

**Tuttnauer** Clinic

LEADING IN  
STERILIZATION  
SINCE

1925

# T-Top Autoclave

Efficiency with maximum reliability



Designed to Meet Your Healthcare  
Clinics' Everyday Sterilization Needs

# Affordable and reliable for healthcare clinic sterilization

## Tabletop Medical Class B Autoclave From The World's Premium Manufacturer

 Efficient Line  Reliable Performance  Plug & Play  Unique User Interface

**T-Top**, a 22 liter tabletop autoclave, was developed specifically for medical and dental clinics of small to medium practices. Designed with simplicity and economy in mind, this tabletop autoclave covers the fundamental needs for your general clinic sterilization with the aim of increasing the productivity of your practice. **T-Top** offers the universal capacity, handed to you, in an affordable and reliable solution you can count on for managing your daily practice sterilization process workload. **T-Top** is the perfect balance between performance, ergonomics, technology & usability. Resulting in a strong and efficient autoclave which perfectly meets the requirements of all standard practices.



# Sterilizer with high performance

Complete solution for clinic instrument reprocessing.

**Making Your Process Easier, Safer and Affordable.**

User friendly advanced technology with Wi-Fi connectivity and intuitive colour touch screens.



## Plug and Play

**No dedicated installation connections needed, simply a power source**

- ◆ Multicolour user-friendly touch screen with intuitive icons
- ◆ Easily start and run programs
- ◆ Compact and ergonomic, designed to fit into your space



## Traceability & Connectivity

**Advanced Technology that monitors & traces your process**

- ◆ 2 USB ports
- ◆ Built-in Wi-Fi connection
- ◆ R.P.C.R



## Reliable Performance

**Quality product that will do all you need and expect**

- ◆ Easy to operate, maintain and use
- ◆ Internal memory - storing up to 500 cycles
- ◆ Advanced technology to deliver longevity and total reliability



## Innovation Best in class technology

- ◆ Programming and customization functions
- ◆ Specially designed user interface to speed up and optimize daily workload
- ◆ Time saver setting to set tests and sterilization cycles to start at any time



## Safety & Regulation

**Meets strictest international standards.**

# Designed for your ease of use

A blend of perfect performance and functionality to **make your user experience intuitive and hassle free.**

## Sterilization at the Click of a Button


Just select the required program and press "start", a click and go feature to help speed up and optimize your instrument reprocessing workload.



# T-Top simplifying Your sterilization process



## Documentation and Traceability

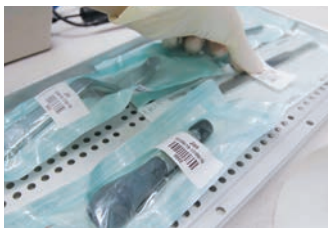

- ◆ Advanced documentation and traceability options, including barcode
- ◆ Personalized username and password








Printouts for saving hard copies (optional printer)



Sophisticated data analysis capabilities with RPCR software

# T-Top


## Sterilization cycle programs

Icon	Program name	Sterilization time	Load type	Type of use
	Unwrapped 134°C	4 min		For immediate use
	Wrapped 134°C	4 min		For storage
	Unwrapped 121°C	20 min	Solid / porous load	For immediate use
	Wrapped 121°C	20 min		For storage
	Prion 134°C	18 min		For storage
Test cycles	Bowie and Dick/Helix/ Vacuum test	Chemical indicator in a product challenge device Periodic testing as referred to in ISO 17665-1		

# T-Top

## Technical specifications

	T-Top 10"
Chamber size	249 (Ø) x 450 (D) mm
Volume	22L
Usable chamber space	75% (~17L)
Overall dimensions	585 (D) x 462 (W) x 460 (H) mm
Max # of trays	5
Tray dimensions	370 (D) x 180 (W) x 15 (H) mm
Weight	46 kg
Connections	1 Ethernet port, 2 USB ports, Wi-Fi
Water filling	manual



**Designed to meet your  
healthcare clinics'**

everyday  
sterilization  
needs

# Sterilization & disinfection consumables

Tuttnauer offers a wide range of indicators to support your sterilization & disinfection process. Within our wide range of products, **we have the fastest most affordable biological indicator, providing an answer within 20 Minutes.**

## Biological Indicators



STEAM  
WTL198-0072



STEAM  
WTL198-0058



STEAM  
WTL198-0067



FLUORESCENCE  
INCUBATOR  
WTL198-0057



STEAM  
WTL198-0056



INCUBATOR  
PLZ198-0005

Fluorescence Super and Ultra Rapid readout

Conventional 24-hour readout

## Chemical Indicators



WTL198-0053  
Indicator tape  
for steam



WTL198-0055  
Indicator for steam



WTL198-0068  
Indicator for steam



WTL 198-0062 (18 min)  
WTL 198-0063 (3.5 min)  
Indicators for steam



WTL198-0087  
Flat rolls for steam  
sterilization



WTL198-0113  
Pouches for steam  
sterilization







### Ultrasonic Cleaner

This quiet yet highly effective cleaner uses soundwaves to rid instruments of hard deposits.



### Sealing Machine

To ensure effective, quality and long-lasting sterilization until the instruments are reused.



### Water Distiller

Small enough to fit on a counter; Smart enough to distill water for sterilization.



### Washer TIVA2

Tiva 2 Washer Disinfector  
Provides a flexible turnkey solution for washing, disinfecting and drying diverse medical instruments for a complete decontamination cycle.



### Washer TIVA8

Tiva 8 Washer Disinfector  
Designed to simplify your work by eliminating manual washing and reducing your manpower.



# Safety

Strict safety regulations require all sterilization processes to be documented, archived and saved.

**T-Top** offers a range of methods for documenting and saving sterilization cycles

## Standards and regulations

**T-Top** meets the following International standards

### CE MDR:

- ◆ Certificate number 3903015CE01
- ◆ Valid until 01-03-2028

### Electromagnetic compatibility:

IEC 61326-1

### Electrical safety:

IEC 61010-1, 61010-2-040

### Verification & validation functional tests:

EN 13060

### Software validation:

EN 62304, FDA guidance

### Quality Management System:

- ◆ ISO 9001:2015
- ◆ ISO 13485:2016

### Pressure Vessel:

PED: 2014/68/EU

# Your clinic's infection control success is backed by 95 years of experience of global leadership

Renowned global leaders in infection control solutions with more than

**95** years experience

Products distributed to clinics, hospitals and research institutions in more than

**140** countries

## International Sales and Marketing

E-mail: [info@tuttnauer.com](mailto:info@tuttnauer.com)  
[www.tuttnauer.com](http://www.tuttnauer.com)

## Tuttnauer Europe B.V.

Hoeksteen 11, 4815 PR  
PO Box 7191, 4800 GD Breda  
The Netherlands  
Tel: +31 765 423 510  
Fax: +31 765 423 540  
E-mail: [info@tuttnauer.nl](mailto:info@tuttnauer.nl)

## Tuttnauer USA Co.

25 Power Drive,  
Hauppauge, NY 11788  
Tel: +800 624 5836, +631 737 4850  
Fax: +631 737 0720  
E-mail: [info@tuttnauerUSA.com](mailto:info@tuttnauerUSA.com)

Distributed by:

  
Innovation · Legacy · Partnership



Learn from our experts

Join our blog [tuttnauerusa.com/blog](http://tuttnauerusa.com/blog)